KRUUSE SafeVet® BANDAGE

Scan QR code and see
how-to video about
KRUUSE SafeVet®
Bandage

Designed in Denmark
For veterinary use only

160718
160719
160720

KRUUSE SafeVet® BANDAGE

GB High-quality cohesive flex bandage

This bandage is suitable for securing dressings, padding materials, and for providing controlled compression and support during treatment or recovery. The cohesive nature of the material allows it to adhere to itself without sticking to skin, fur or feathers, making it easy to apply and remove without causing discomfort. It is strong, breathable, flexible and skin-friendly, ensuring both performance and animal comfort.

Choose an appropriate bandage width and length for the size of the animal and treatment site. Start bandaging at a point below the affected area and work upward, overlapping each layer by approximately half the bandage's width. Stretch the bandage gently to apply even compression, taking care not to wrap too tightly, as this can impair circulation. Once applied, press gently along the bandage to activate cohesion between layers and confirm that the bandage is secure and evenly distributed. Regularly monitor the bandaged area for signs of swelling, discoloration or discomfort, especially in limbs or distal areas where circulation is more vulnerable. Adjust or replace the bandage as needed.

The bandage can be removed by gently unrolling or carefully cutting with bandage scissors. Use caution to avoid injury to the animal's skin or coat during removal. Dispose of used bandages in accordance with local regulations.

Do not re-use, as this may compromise hygiene and performance.

Store in a cool, dry place away from direct sunlight, heat and moisture to preserve product integrity.

The **KRUUSE SafeVet® Bandage** features a distinctive **protected design pattern**, that visualizes the actual stretch level. The visual cue functions as an early warning indication, to reduce the risk of stasis. Bandages should be applied with maximum 50% stretch to avoid the risk of stasis.

KRUUSE SafeVet® Bandage – for safe and gentle treatment

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© Design Protected No. 015080152
JØRGEN KRUUSE A/S, Havretoften 4, 5550 Langeskov, Denmark