

# BUSTER Body Sleeves Front Legs

The unique Body Sleeves are available in a version for front legs and a version for hind legs depending on the area affected on the dog. The Body Sleeves can be used for both cats and dogs. Please see the guidelines for sizes.

## Features and Benefits of the BUSTER Body Sleeves

- Comfortable, soft and flexible
- Breathable material
- Freedom of movement
- Easy fitting and removal
- Made from polyester/viscose/elastane
- Easy-to-cut non-fraying fabric



## BUSTER Body Sleeves Front Legs


- The extra sleeve fabric can be folded around the paw of the dog in case of a paw injury. Please use an extra anti-slid paw protection for the safety of the dog.
- The remaining sleeve fabric can be cut off in a suitable length for the individual dog



# BUSTER Body Sleeves Front Legs

Guideline for Sizes

The BUSTER Body Sleeves for front legs is available in 8 sizes.

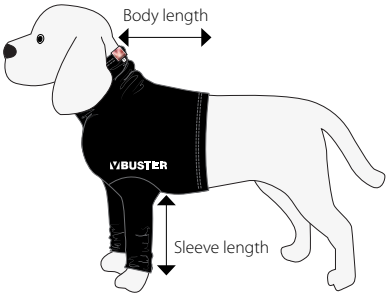

Cat. No	Description	Sleeve Length	Body Length	Suitable for
274046	BUSTER Body Sleeves, front legs XXXS	21 cm	24 cm	Small Chihuahua, kitten
274032	BUSTER Body Sleeves, front legs, XXS	25 cm	30 cm	Chihuahua, Toy poodle, Shih Tzu, Cat
274033	BUSTER Body Sleeves, front legs, XS	29 cm	36 cm	Yorkshire Terrier, Lhasa Apso, Toy poodle, Cat
274034	BUSTER Body Sleeves, front legs, S	33 cm	42 cm	Maltese, Pekingese, French Bulldog, Pug, Terrier
274035	BUSTER Body Sleeves, front legs, M	37 cm	46 cm	English Cocker Spaniel, Samoyed, Portuguese Podengo
274036	BUSTER Body Sleeves, front legs, L	41 cm	50 cm	Schnauzer, Poodle, Fox Terrier, Chow-Chow
274037	BUSTER Body Sleeves, front legs, XL	45 cm	57 cm	Collie, Appenzeller, Boxer, Labrador Retriever
274038	BUSTER Body Sleeves, front legs, XXL	49 cm	64 cm	Rottweiler, Golden Retriever, Herdershond

Instructions for use

1. Roll up the Body Sleeves between the hands having the KRUUSE logo at the top towards your own body
2. Pull the Body Sleeves over the dog's head having the KRUUSE logo on top of the dog's body
3. Pull the Body Sleeves over the dog's body
4. Roll up one of the sleeves with your hand from the bottom of the sleeve upwards
5. The dog's leg is lifted up and inserted into the sleeve. Your hand inside the sleeve holds the dog's paws avoiding the claws etc. to get caught. With your other hand you pull up the sleeve
6. Pull up the sleeve entirely till the elbow joint of the dog
7. Repeat bullet 4-6 on the second leg
8. Pull up the Body Sleeves around the body
9. Put your hand along the dog's back inside the Body Sleeve, from the back up to the neck, lift up enabling the Body Sleeve to fit correctly into the elbow joint and under the body of the dog

It is important that the sleeves are pulled up well so that the sleeve seam fully stretches the dog's body and snuggles the dog. Adjust the length of the sleeves, the sleeves can be folded to suit the dog or the remaining fabric can be cut off.

The BUSTER Body Sleeves are washable at 40°C



Instructions for use  
Scan QR code to see the video