

## BUSTER Body Sleeves Hind Legs

The unique Body Sleeves are available in a version for front legs and a version for hind legs depending on the area affected on the dog.

The Body Sleeves can be used for both cats and dogs. Please see the guidelines for sizes.

### Features and Benefits of the BUSTER Body Sleeves

- Comfortable, soft and flexible
- Breathable material
- Freedom of movement
- Easy fitting and removal
- Made from polyester/viscose/elastane
- Easy-to-cut non-fraying fabric



Let the dog keep the Body Sleeves on when nature calls, and easily cut a customised slit or hole in the non-fraying fabric.

### BUSTER Body Sleeves Hind Legs


- The extra sleeve fabric can be folded around the paw of the dog in case of a paw injury. Please use an extra anti-slid paw protection for the safety of the dog.
- The remaining sleeve fabric can be cut off in a suitable length for the individual dog



# BUSTER Body Sleeves Hind Legs

Guideline for Sizes

The BUSTER Body Sleeves for hind legs is available in 8 sizes.

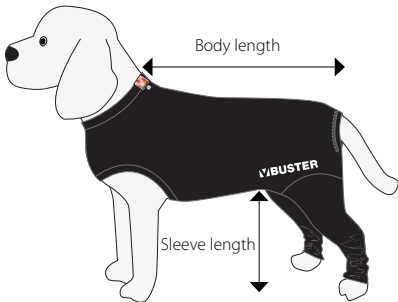

Cat. No	Description	Sleeve Length	Body Length	Suitable for
274047	BUSTER Body Sleeves, hind legs, XXXS	27 cm	28 cm	Small Chihuahua, kitten
274039	BUSTER Body Sleeves, hind legs, XXS	31 cm	36 cm	Chihuahua, Toy poodle, Shih Tzu, Cat
274040	BUSTER Body Sleeves, hind legs, XS	35 cm	45 cm	Yorkshire Terrier, Lhasa Apso, Toy poodle, Cat
274041	BUSTER Body Sleeves, hind legs, S	39 cm	50 cm	Maltese, Pekingese, French Bulldog, Pug, Terrier
274042	BUSTER Body Sleeves, hind legs, M	43 cm	55 cm	English Cocker Spaniel, Samoyed, Portuguese Podengo
274043	BUSTER Body Sleeves, hind legs, L	47 cm	61 cm	Schnauzer, Poodle, Fox Terrier, Chow-Chow
274044	BUSTER Body Sleeves, hind legs, XL	51 cm	71 cm	Collie, Appenzeller, Boxer, Labrador Retriever
274045	BUSTER Body Sleeves, hind legs, XXL	55 cm	81 cm	Rottweiler, Golden Retriever, Herdershond

Instructions for use

1. Roll up the Body Sleeve and one sleeve between the hands, the KRUUSE logo must be on top of the dog’s back
2. Put one of the dog’s leg legs into the sleeve and pull the sleeve entirely till the dog’s groin  
Pull the tail through the tail hole at the end of the Body Sleeve
3. Roll up the other sleeve and put the dog’s second leg into the sleeve, pull up the sleeve entirely till the groin
4. Drag the Body Sleeve over the body of the dog
5. Lift up one front leg and pull through the leg hole in the Body Sleeve
6. Lift up the second front leg and pull through the leg hole in the Body Sleeve
7. Drag the Body Sleeve all the way up to the neck of the dog. Put your hand inside the  
Body Sleeve along the dog’s back from the dog’s ears to the tail - and pull up the Body Sleeve in place

It is important that the sleeves are pulled up all the way so that the seam follows the body of the dog (in the groin).  
Adjust the length of the sleeves of the hind legs, the sleeves can be folded to suit the dog or the remaining fabric can be cut off.



Instructions for use  
Scan QR code to see the video