

BUSTER Body Suit & Body Sleeves for Dogs & Cats

The soft and comfortable BUSTER Body Suit and Body Sleeves are used as protection and comfort after surgery or in case of skin disease.

It is an ideal solution for patients recovering from surgery or minor injuries or have tender areas caused by excessive licking, biting or chewing.

BUSTER Body Suits are perfect for

- Postoperative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues
- In season

BUSTER Body Sleeves are perfect for

- Postoperative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments



BUSTER Body Suit

The BUSTER Body Suit is available in a version for dogs and a version for cats.

Features and Benefits of the BUSTER Body Suit



- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Press studs for more security
- Made from cotton/elastane

**DOUBLE FABRIC
UNDER THE ABDOMEN**





BUSTER Body Suit Classic for Dogs

- Double fabric under the abdomen enabling pads to be inserted in case of fluid loss, incontinence or when the animal is in season.
- Undo press studs and touch fasteners to pull back the fabric. Roll the fabric under the abdomen and above the back when nature calls.
- It is suitable for bitches and male dogs

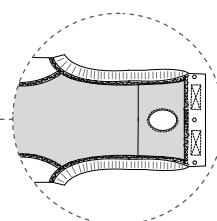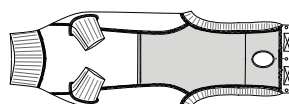

BUSTER Body Suit EasyGo for Cats


- Double fabric under the abdomen enabling pads to be inserted in case of fluid loss
- Body Suit EasyGo for cats with extended tail opening.
- The EasyGo version helps the cat when nature calls
- It is suitable for female cats and tomcats

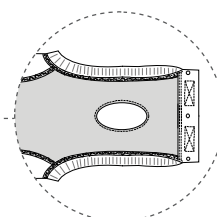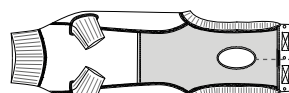

BUSTER Body Suit

Instructions for Use


- 1. Insert the head into the Body Suit
- 2. Insert each front leg separately into the openings provided
- 3. Pull the Body Suit as far as possible over the body
- 4. Put the closing flap between the rear legs and insert the tail through the tail opening
- 5. Attach the closing flap on the touch fasteners on the loin and secure the press studs

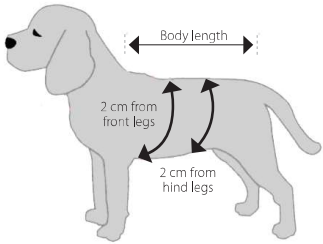

Guideline for Sizes

The BUSTER Body Suit Classic for dogs is available in 9 sizes.
Size measured from collar to base of the tail.

Cat. No	Size	Body length	Chest circumference (2 cm from front legs)	Abdomen circumference (2 cm from hind legs)	Suitable for
273968	XXXS	26 cm	30 cm	26 cm	Small Chihuahua
273969	XXS	33 cm	38 cm	34 cm	Chihuahua, Toy poodle
273970	XS	39 cm	46 cm	42 cm	Yorkshire, Terrier, Lhasa, Toy poodle
273971	S	42 cm	54 cm	50 cm	Malteser, Pekingese, French Bulldog, Mops, Terrier
273972	M	46 cm	58 cm	52 cm	English Cocker Spaniel, Samojed, Portuguese Podengo
273973	L	53 cm	62 cm	56 cm	Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow
273974	XL	63 cm	70 cm	64 cm	Collie, Appenzeller, Boxer, Schnauzer, Labrador
273975	XXL	74 cm	78 cm	70 cm	Rottweiler, Golden Retriever, Herdershond
273976	XXXL	76 cm	86 cm	76 cm	Saint Bernhard, Newfoundlander

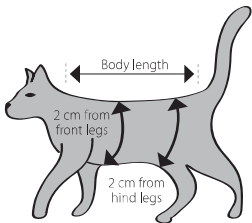

The BUSTER Body Suit EasyGo for cats is available in 4 sizes.
Size measured from collar to base of the tail.

Cat. No	Size	Body length	Chest circumference (2 cm from front legs)	Abdomen circumference (2 cm from hind legs)	Suitable for
273960	XXXS	27.5 cm	36 cm	30 cm	Young cat (small kitten)
273961	XXS	33 cm	42 cm	36 cm	Young cat (kitten)
273962	XS	38.5 cm	46 cm	40 cm	Adult cat
273963	S	43.5 cm	52 cm	46 cm	Adult cat (large breed)

The BUSTER Body Suit is washable at 40°C

